
YOUR GUIDE FOR A BETTER PERIOD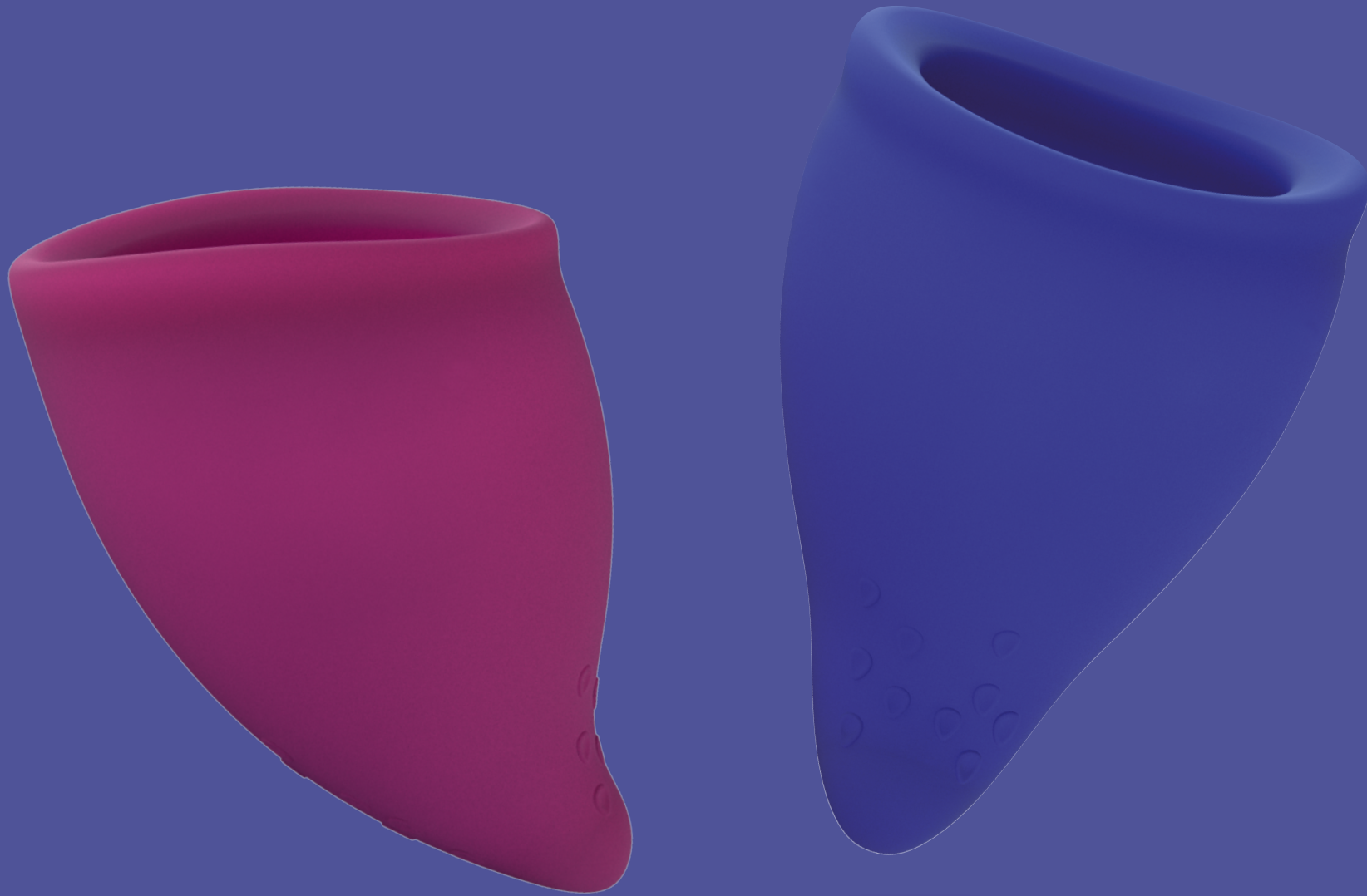

FUN CUP

FUN CUP is a menstrual cup made of medical silicone and is only suitable for vaginal use. Please read the instructions for use and keep them as a reference. Learn about the medical risks associated with using menstrual products from your Healthcare Provider. Wash your hands thoroughly with a mild antibacterial soap and water before inserting or removing the FUN CUP. Boil the FUN CUP in tap water for at least 5 minutes before and between uses.

CLEANING

For initial use

1. Fill a small pot with tap water and bring to a boil. Submerge the FUN CUP completely into the boiling water for at least 5 minutes.
2. Empty the pot and let the cup return to room temperature.
3. Wash your hands thoroughly with warm water and a mild antibacterial soap. Hands should be dried well afterwards.
4. Rub the FUN CUP a little mild water-based, oil-free soap and rinse with tap water.
5. Dry off the cup using a clean lint-free towel. Now it is ready to insert.

For repeat usage

1. Wash your hands thoroughly with warm water and a mild antibacterial soap. Hands should be dried well afterwards.
2. Rub the FUN CUP a little mild water-based, oil-free soap and rinse with tap water.
3. Dry off the cup using a clean lint-free towel.
Now it is ready for re-insertion.

In between cycles

1. Wash your hands thoroughly with warm water and a mild antibacterial soap. Hands should be dried well afterwards.
2. Rub the FUN CUP a little mild water-based, oil-free soap and rinse with tap water.
3. Fill a small pot with tap water and bring to a boil. Submerge the FUN CUP completely into the boiling water for at least 5 minutes. Pay particular attention to the hard-to-reach holes they may require some extra cleaning.
4. Dry off the cup using a clean lint-free towel.
5. Store your FUN CUP in the bag provided. Check that the bag is clean and dry before storing. If needed clean the bag with tap water and a little mild soap then air dry.
6. Store the bag with products in a dry place that is protected from direct sunlight.

INSERTION

1. Wash your hands thoroughly with warm water and a mild antibacterial soap. Hands should be dried well afterwards
2. Before inserting the FUN CUP check for defects, surface film or foul odors. If your menstrual cup has any defects or foul odor, do not use it.
3. Fold the FUN CUP (C-fold or Pinch-fold) as diagrams illustrate.
4. Insert it into your vagina.

5. Push the FUN CUP in until you have overcome a natural resistance and then release.
6. The FUN CUP will then open covering your cervix. To create a vacuum-like seal twist the cup holding (either holding the sides or the firm silicone tip) while bearing down with your pelvic muscles. Give it a gentle tug to make sure it's created a seal. If it has, you should be able to feel a little resistance.



REMOVAL

1. Wash your hands with warm water and a mild antibacterial soap properly. Hands should be dried well afterwards.
2. We recommend removing the FUN CUP every 6 hours to empty and clean it properly. Should not be worn longer than 8 hours. Remove and clean more often if you have heavy periods.
3. Use your pelvic muscles to push the cup down, and then pinch it together with two fingers to release the seal or press a finger against the top of the FUN CUP to release the seal.
4. Carefully pull out the cup with the opening facing upwards and empty it into the toilet.
5. NEVER use a foreign object such as tweezers to remove the FUN CUP!

NOTABLE INFORMATION

- TSS (Toxic Shock Syndrome) is a very rare condition that can occur with menstrual products and can be fatal without treatment. Symptoms of TSS include sudden fever, chills, nausea, vomiting, muscle pain, drop in blood pressure, dizziness, and drowsiness. If the symptoms appear, remove the FUN CUP immediately and consult your Healthcare Provider. If TSS has already occurred or if there are injuries and infections in the genital area, the use of the FUN CUP and other inserted menstrual products is not recommended. We also recommend consulting your Healthcare Provider before using a menstrual product for the first time.
- Boil the FUN CUP in tap water for at least 5 minutes or discard the FUN CUP if it fell in a public toilet or toilet containing urine or feces.
- Don't use the FUN CUP at the time of a yeast or bacterial infection and consult your doctor if necessary.
- FUN CUP is for personal use only and should not be shared with other persons despite cleaning measures to avoid the transmission of germs.

For more languages, check:
www.FUNFACTORY.com/manuals

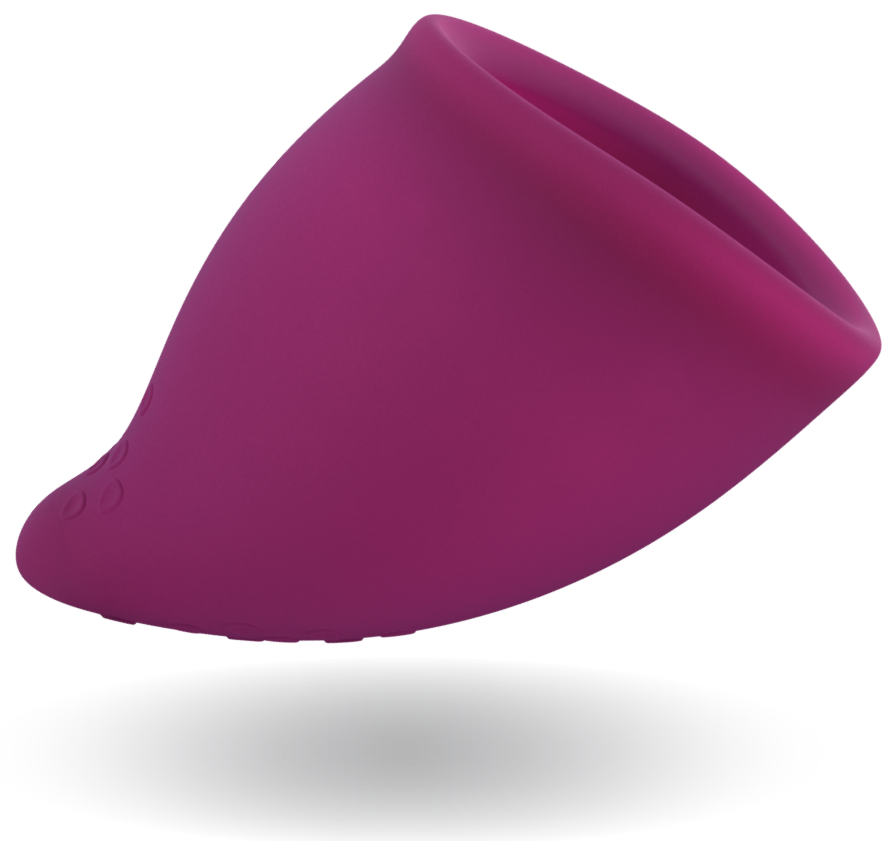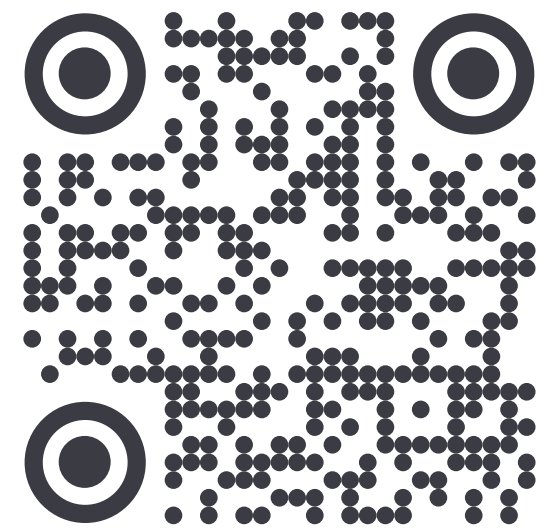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FUN FACTORY Germany GmbH